

Workstation on Wheels (WoW)/Medical Delivery Care (MDC) Cleaning and Disinfection Guide

Perform hand hygiene before & after contact with WOW/MDC, the patient, or their environment.

VCH-approved products:

- Accelerated Hydrogen Peroxide 0.5 % – AHP (Brand Name Accel INTERvention Wipes)

ALL SURFACES EXCEPT SCREENS

- Allow the chemical to dwell for product-stated wet contact time (1 minute) Followed by air-drying;
- If any concerns regarding residual product use a clean well squeezed-out cloth with plain water to remove any remaining residue.

- Alcohol 79% (Brand Name ULTRA Swipes) **SCREENS ONLY**

- Allow the chemical to dwell for product-stated wet contact time (1 minute) Followed by air-drying;
- If any concerns regarding residual product (i.e., streaking on screens), use a clean well squeezed-out cloth with plain water to remove any remaining residue.



Cleaning and disinfection is a two-step process:

- Step 1: Clean** with a new wipe(s) or cloth using friction (rub/scrub motion) to remove any foreign matter (e.g. dust, soil, food, feces, blood, sputum) and then immediately go to step 2.
- Step 2: Disinfect** using friction (rub/scrub motion) with new wipe(s). To complete the disinfection process, item must be wet for 1 minute. Allow item to **air-dry** to complete disinfection process.

Detailed WOW/MDC Cleaning & Disinfection by Component

Equipment	Frequency	Details	Accountability	Product
Tangent Screen	<ul style="list-style-type: none"> Daily and whenever visibly soiled After direct or indirect contact with a patient On exit from room on Additional Precautions (isolation) Deep clean* monthly 	Includes screen (exterior plastic should be cleaned and disinfected with AHP) <i>Avoid exposed fan area</i> If any concerns regarding residual product, use a	<ul style="list-style-type: none"> Direct Care Users EVS daily and on discharge if left in the patient room 	Alcohol <i>*Damp but not saturated</i>



Tangent Batteries		clean well squeezed-out cloth with plain water to remove residue	<ul style="list-style-type: none"> EVS monthly deep clean 	
	<ul style="list-style-type: none"> Between use and whenever visibly soiled 	Plastic area, avoid copper strips	Direct Care Users	Accelerated Hydrogen Peroxide
Tangent Battery Charger	<ul style="list-style-type: none"> Daily and whenever visibly soiled 	Exterior only <i>* Stored on the unit, not attached to the WOW</i> If interior cleaning is required, unplug the device and use an air pressure spray can to remove dust and debris	EVS BioMed (interior)	Accelerated Hydrogen Peroxide
Ergotron Cart (WOW/MDC)	<ul style="list-style-type: none"> Daily and whenever visibly soiled After direct or indirect contact with a patient On exit from room on Additional Precautions (isolation) Deep clean* monthly 	Includes work surface, stand, entire base, code pin pad and exterior accessories	<ul style="list-style-type: none"> Direct Care Users EVS daily and on discharge if left in the patient room EVS monthly deep clean 	Accelerated Hydrogen Peroxide
Ergotron Cart Drawers, if applicable (WOW/MDC)	<ul style="list-style-type: none"> Daily and whenever visibly soiled After direct or indirect contact with a patient On exit from room on Additional Precautions (isolation) Deep clean* monthly 	Includes interior and exterior drawers <i>*interior drawers cleaned as needed by direct care users (i.e., when drawer emptied between patients or between shifts) & biweekly by EVS, drawers must be emptied for deep clean</i>	<ul style="list-style-type: none"> Direct Care Users EVS daily and on discharge if left in the patient room EVS monthly deep clean 	Accelerated Hydrogen Peroxide
Seal Shield Keyboard	<ul style="list-style-type: none"> Daily and whenever visibly soiled After direct or indirect contact with a patient On exit from room on Additional Precautions (isolation) Deep clean* monthly 	Entire keyboard Cable – damp wipe only	<ul style="list-style-type: none"> Direct Care Users EVS daily and on discharge if left in the patient room EVS monthly deep clean 	Accelerated Hydrogen Peroxide Seal Shield keyboard is waterproof and fully submersible if needed for deep cleaning
Seal Shield Mouse	<ul style="list-style-type: none"> Daily and whenever visibly soiled 	Entire mouse Cable – damp wipe only	<ul style="list-style-type: none"> Direct Care Users 	Accelerated Hydrogen Peroxide



Code Barcode Reader	<ul style="list-style-type: none"> After direct or indirect contact with a patient On exit from room on Additional Precautions (isolation) Deep clean* monthly 		<ul style="list-style-type: none"> EVS daily and on discharge if left in the patient room EVS monthly deep clean 	Seal Shield mouse is waterproof and fully submersible if needed for deep cleaning
	<ul style="list-style-type: none"> Daily and whenever visibly soiled After direct or indirect contact with a patient On exit from room on Additional Precautions (isolation) Deep clean monthly* 	<p>Entire barcode reader</p> <p>Avoid metal strips near scanning screen</p>	<ul style="list-style-type: none"> Direct Care Users EVS daily and on discharge if left in the patient room EVS monthly deep clean 	Accelerated Hydrogen Peroxide
Code Reader Holder and Charger	<ul style="list-style-type: none"> Daily and whenever visibly soiled After direct or indirect contact with a patient On exit from room on Additional Precautions (isolation) Deep clean* monthly 	<p>Exterior Only</p> <p>If interior cleaning is required, unplug the device and use an air pressure spray can to remove dust and debris</p>	<ul style="list-style-type: none"> Direct Care Users EVS daily and on discharge if left in the patient room EVS monthly deep clean Biomed (Interior) 	Accelerated Hydrogen Peroxide
FESR Olympic Microphone DR-1200	<ul style="list-style-type: none"> Daily and whenever visibly soiled After direct or indirect contact with a patient On exit from room on Additional Precautions (isolation) Deep clean* monthly 	Exterior damp wipe only	<ul style="list-style-type: none"> Direct Care Users EVS daily and on discharge if left in the patient room EVS monthly deep clean 	<p>Accelerated Hydrogen Peroxide</p> <p><i>*Damp but not saturated</i></p> <p><i>Do not use alcohol (e.g. Ultra Swipes Plus)</i></p>

Deep clean includes entire WOW device: Wheels, stand and underside of workstation, mouse pad, mouse tray, keyboard, cables, workstation top, gel cup holder, FESR microphone, Tangent screen and exterior, Accel INTERVention wipes holder and dispenser, hand sanitizer pump and holder, drawers (exterior and interior) and Code Pad, if applicable to the device.

Ambulatory & Community settings deep clean:
 Monthly for clinics/programs with invasive procedures
 Quarterly for clinics/programs



REFERENCES:

1. Seal Shield. (2017). [Facility Managers & Environmental Services](#).
2. Ergotron. (2016). General Cleaning Tips. Retrieved from:



SV Cart Cleaning Tips
(Letter form).pdf

3. Code Revolutionizing Barcode Reading. (2016). [Code Reader™ 2600](#).
4. Tangent. (2017). [Medical Grade Computers](#).
5. Tangent. (2016). Tangent HSSBC Presentation (5-4-16). Retrieved from:



Tangent HSSBC
Presentation (5-4-16)

6. Tangent. (2016). Medix Cleaning Recommendation. Retrieved from:



Tangent Medix
Cleaning recommend

7. Olympus DR-1200 [Manual : Safety Precautions](#).
8. Olympus. (2017). Cleaning Process for Olympus DR-1200 Microphone. Retrieved from:



FINAL_PCDOCS-#71
112-v1-OAI_CPG_Prc

