

Hand-held Tono-Pen Cleaning and Disinfection Process

1. Remove the Tono-Pen from storage case.
2. Perform hand hygiene and put on disposable gloves.
3. Remove the storage Ocu-Film + tip cover from the Transducer Assembly, if in place.
4. Visually inspect the Transducer Assembly (metal part of the Tono-Pen) for cracks, chips or other irregularities. Do not use if these conditions are present.



5. Using Clorox disinfectant wipe, rub the housing cover (red arrow) then the sides of the tip (yellow arrow), **NOT THE TIP** of the Tono-Pen. Then wait to dry for one minute.



6. Slide an Ocu-Film + Tip Cover onto the Transducer Assembly (metal part of the Tono-Pen) until the ridge is seated, taking care not to apply the tip cover too tightly or too loosely. Make certain that the rubber is flat across the tip, but not taut.





7. Remove gloves & perform hand hygiene. Put on new gloves if required.
8. Complete the patient exam. **DO NOT** use the Tono-Pen on a patient without a new tip cover installed.
9. After using the hand-held Tono-Pen, remove and discard the Ocu-Film + tip cover from the transducer assembly.
10. Remove gloves and perform hand hygiene.
11. Put on new disposable gloves and repeat Step 5.
12. When storing Tono-Pen, cover the tip with the tip cover to prevent dust and contaminant.
13. Remove gloves and perform hand hygiene.
14. Return the Tono-Pen to the case.

