



How To Use Guide

Nasopharyngeal Flocked Swabs and Universal Transport Medium (UTM)

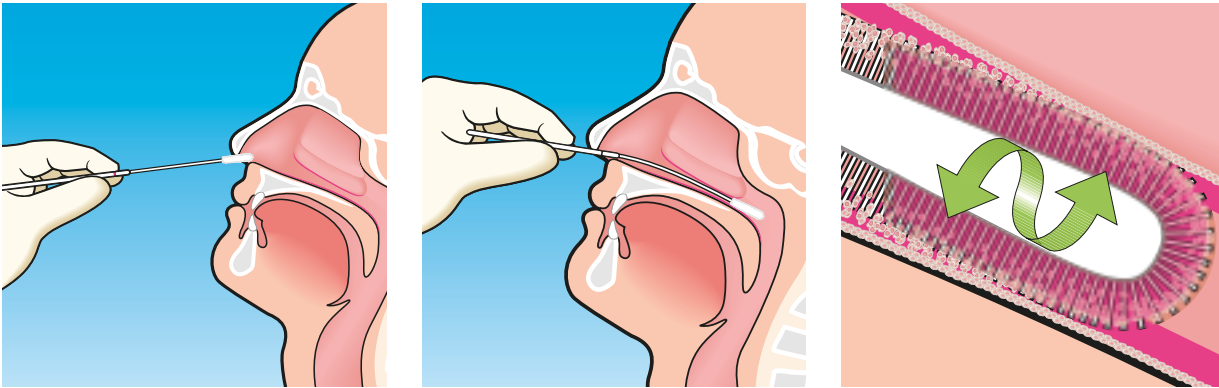

1. Gently insert the swab along the nasal septum just above the floor of the passage to the nasopharynx until resistance is met
2. Rotate the swab gently against the nasopharyngeal mucosa for 10 - 15 seconds then gently remove swab

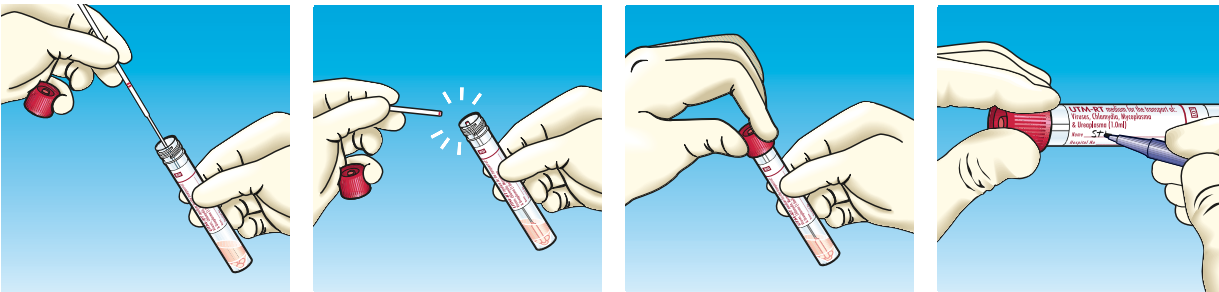

3. After the swab is removed from the patient place it into the tube of UTM transport medium all the way to the bottom of the tube
4. Holding the swab shaft close to the rim of the tube, break the applicator shaft at the colored breakpoint indication line. **Hold the tube opening away from your face.**

ORDERING INFORMATION:

Individual Components:

- 330C Universal Transport Medium (UTM) tube for Viruses, Chlamydia, Mycoplasma and Ureaplasma
- 503CS01 Flexible Minitip Flocked Swab

Collection Kit:

- 305C Kit comprises UTM tube plus Flexible Minitip Flocked Swab



Copan Diagnostics Inc.

26055 Jefferson Ave. • Murrieta, California 92562 USA

Toll Free: (800)216-4016 (US & Canada)

Phone: (951)696-6957 Fax: (951)600-1832

E-Mail: info@copanusa.com

www.copanusa.com

00609VPD

COPAN
flocked SWABS

UTM
UNIVERSAL TRANSPORT MEDIUM