

## Cleaning and Disinfection of Eye Protection



### Goggles, Safety Glasses, or Face Shield:

Inspect eye protection for defects and discard if any found. If face shield comfort foam becomes visibly soiled, discard.

- Don (put on) gloves.
- Using an Accel wipe or Ultra Swipe (or approved alternate product), clean (first step) from the inside to the outside of the face shield or goggles, then with a new wipe disinfect (second step). **Allow 1 minute wet contact time.**
- Wipe or rinse with clean water to remove residue.
- Allow to fully dry (air dry or wipe with clean absorbent towel).
- Remove gloves and perform hand hygiene.

