

How to Clean and Disinfect a Workstation On Wheels (WOW)/Medication Delivery Cart (MDC)

WHAT	WHO
Quick: ● All high-touch surfaces: <ul style="list-style-type: none"> barcode scanner/holder height adjustment handle FESR microphone (if used) medication drawer handles med prep/work surface keyboard & mouse pin pad 	Direct Care User
Complete: ● ◆ Quick clean, plus base and screen.	Direct Care User
Deep: ● ◆ ▲ Complete clean, plus all remaining surfaces.	Environmental Services



WHEN	Quick	Complete	Deep
Routine – daily/shift change		◆	
Before clean task (e.g. med prep, gathering clean supplies)	●		
Used within 2 meters of a patient/client	●		
Items/equipment from patient environment touched the WOW/MDC	●		
Exiting an isolation room		◆	
Visibly soiled (e.g. small area)		◆	
Heavily soiled (e.g. large area, drawers, crevices, etc.)			▲
Routine – monthly			▲

HOW	Product to Use*	
(2-Step Process) Step 1: Clean with new wipe(s) using friction (rub/scrub). Step 2: Disinfect with new wipe(s). Allow to air dry following manufacturer recommended wet contact (dwell) time for disinfection.	Accelerated Hydrogen Peroxide (all surfaces except screen)  Accel INTERvention™	Alcohol (screen only)  ULTRA Swipes™



*Click link to determine which product is appropriate for your situation and to view important information about required PPE and dwell/contact times.

