

CST WOW (Workstation on Wheels) Device Cleaning and Disinfection Guide

Perform hand hygiene before & after contact with WOW, the patient, or their environment.

VCH-approved products:

- Accelerated Hydrogen Peroxide 0.5 % – AHP (Brand Name Accel INTERVention Wipes) **DO NOT USE ANY OTHER PRODUCTS, AS THEY MAY CAUSE DAMAGE**
 - Allow the chemical to dwell for product-stated wet contact time (1 minute) Followed by air-drying;
 - If any concerns regarding residual product use a clean well squeezed-out cloth with plain water to remove any remaining residue.

- Alcohol 79% (Brand Name ULTRA Swipes) **SCREENS ONLY**
 - Allow the chemical to dwell for product-stated wet contact time (1 minute) Followed by air-drying;
 - If any concerns regarding residual product (i.e., streaking on screens), use a clean well squeezed-out cloth with plain water to remove any remaining residue.



Cleaning and disinfection is a two-step process:

- **Step 1: Clean** with a new wipe(s) or cloth using friction (rub/scrub motion) to remove any foreign matter (e.g. dust, soil, food, feces, blood, sputum) and then immediately go to step 2.
- **Step 2: Disinfect** using friction (rub/scrub motion) with new wipe(s). To complete the disinfection process, item must be wet for 1 minute. Allow item to **air-dry** to complete disinfection process.

Detailed WOW Device Cleaning & Disinfection by Component

Equipment	Frequency	Details	Accountability	Product
Tangent Screen	<ul style="list-style-type: none"> • Daily and whenever visibly soiled • After direct or indirect contact with a patient • On exit from room on Additional Precautions (isolation) • Deep clean* bi-weekly 	<p>Includes screen and exterior plastic</p> <p><i>Avoid exposed fan area</i></p> <p>If any concerns regarding residual product, use a clean well squeezed-out cloth with plain water to remove any remaining residue</p>	<ul style="list-style-type: none"> • Direct Care Users • EVS: Bi-weekly deep clean and daily/on discharge if left in the patient room 	<p>Alcohol</p> <p><i>*Damp but not saturated</i></p>

Tangent Batteries	<ul style="list-style-type: none"> Between use and whenever visibly soiled 	Plastic area, avoid copper strips	Direct Care Users	Accelerated Hydrogen Peroxide
Tangent Battery Charger	<ul style="list-style-type: none"> Daily and whenever visibly soiled 	Exterior only * <i>Stored on the unit, not attached to the WOW</i> If interior cleaning is required, unplug the device and use an air pressure spray can to remove dust and debris	EVS BioMed (interior)	Accelerated Hydrogen Peroxide
Ergotron Cart (WOW/MDC)	<ul style="list-style-type: none"> Daily and whenever visibly soiled After direct or indirect contact with a patient On exit from room on Additional Precautions (isolation) Deep clean* bi-weekly 	Includes work surface, stand, entire base, code pin pad and exterior accessories	<ul style="list-style-type: none"> Direct Care Users EVS Bi-weekly deep clean and daily/on discharge if left in the patient room 	Accelerated Hydrogen Peroxide
Ergotron Cart Drawers, if applicable (WOW/MDC)	<ul style="list-style-type: none"> Daily and whenever visibly soiled After direct or indirect contact with a patient On exit from room on Additional Precautions (isolation) Deep clean* bi-weekly 	Includes interior and exterior drawers *interior drawers cleaned as needed by direct care users (i.e., when drawer emptied between patients or between shifts) & biweekly by EVS, drawers must be emptied for deep clean	<ul style="list-style-type: none"> Direct Care Users EVS Bi-weekly deep clean and daily/on discharge if left in the patient room 	Accelerated Hydrogen Peroxide
Seal Shield Keyboard	<ul style="list-style-type: none"> Daily and whenever visibly soiled After direct or indirect contact with a patient On exit from room on Additional Precautions (isolation) Deep clean* bi-weekly 	Entire keyboard Cable – damp wipe only	<ul style="list-style-type: none"> Direct Care Users EVS Bi-weekly deep clean and daily/on discharge if left in the patient room 	Accelerated Hydrogen Peroxide Seal Shield keyboard is waterproof and fully submersible if needed for deep cleaning
Seal Shield Mouse	<ul style="list-style-type: none"> Daily and whenever visibly soiled After direct or indirect contact with a patient On exit from room 	Entire mouse Cable – damp wipe only	<ul style="list-style-type: none"> Direct Care Users EVS Bi-weekly deep clean and daily/on 	Accelerated Hydrogen Peroxide Seal Shield mouse is waterproof and fully submersible if needed for deep

	<ul style="list-style-type: none"> on Additional Precautions (isolation) • Deep clean* bi-weekly 		<ul style="list-style-type: none"> discharge if left in the patient room 	cleaning
Code Barcode Reader	<ul style="list-style-type: none"> • Daily and whenever visibly soiled • After direct or indirect contact with a patient • On exit from room on Additional Precautions (isolation) • Deep clean bi-weekly* 	<ul style="list-style-type: none"> Entire barcode reader Avoid metal strips near scanning screen 	<ul style="list-style-type: none"> • Direct Care Users • EVS Bi-weekly deep clean and daily/on discharge if left in the patient room 	Accelerated Hydrogen Peroxide
Code Reader Holder and Charger	<ul style="list-style-type: none"> • Daily and whenever visibly soiled • After direct or indirect contact with a patient • On exit from room on Additional Precautions (isolation) • Deep clean* bi-weekly 	<ul style="list-style-type: none"> Exterior Only If interior cleaning is required, unplug the device and use an air pressure spray can to remove dust and debris 	<ul style="list-style-type: none"> • Direct Care Users • EVS Bi-weekly deep clean and daily/on discharge if left in the patient room • BioMed (interior) 	Accelerated Hydrogen Peroxide
FESR Olympic Microphone DR-1200	<ul style="list-style-type: none"> • Daily and whenever visibly soiled • After direct or indirect contact with a patient • On exit from room on Additional Precautions (isolation) • Deep clean* bi-weekly 	Exterior damp wipe only	<ul style="list-style-type: none"> • Direct Care Users • EVS Bi-weekly deep clean and daily/on discharge if left in the patient room 	<ul style="list-style-type: none"> Accelerated Hydrogen Peroxide *Damp but not saturated Do not use alcohol (e.g. Ultra Swipes Plus)

**Deep clean includes entire WOW device: Wheels, stand and underside of workstation, mouse pad, mouse tray, keyboard, cables, workstation top, gel cup holder, FESR microphone, Tangent screen and exterior, Accel INTERVention wipes holder and dispenser, hand sanitizer pump and holder, drawers (exterior and interior) and Code Pad, if applicable to the device.*

REFERENCES:

- Seal Shield. (2017). Facility Managers & Environmental Services. Retrieved from: <http://www.sealshield.com/Facility-Managers.html>.
- Ergotron. (2016). General Cleaning Tips. Retrieved from:



SV Cart Cleaning Tips
(Letter form).pdf

- Code Revolutionizing Barcode Reading. (2016). Code Reader™ 2600. Retrieved from:
<http://www.codecorp.com/products.php?id=138>.
- Tangent. (2017). Medical Grade Computers. Retrieved from:
<https://www.tangent.com/healthcare/medical-computers>.

- Tangent. (2016). Tangent HSSBC Presentation (5-4-16). Retrieved from:



Tangent HSSBC
Presentation (5-4-16)

- Tangent. (2016). Medix Cleaning Recommendation. Retrieved from:



Tangent Medix
Cleaning recommenda

- Olympus DR-1200 Manual : Safety Precautions. Retrieved from:
<https://www.manualslib.com/manual/802455/Olympus-Dr-1200.html?page=2#manual>
- Olympus. (2017). Cleaning Process for Olympus DR-1200 Microphone. Retrieved from:



FINAL_PCDOCS-#71
112-v1-OAI_CPG_Prc