

HOW TO CLEAN & DISINFECT A WOW

Cleaning & disinfection is a 2-step process using:

Accel INTERVention
(All surfaces EXCEPT screen)



or

ULTRA Swipes
(screen only)



STEP 1: Clean with new wipe(s) using friction (rub/scrub).

STEP 2: Disinfect with new wipe(s), must stay wet for 1 minute. Allow to air-dry.

Screen

Use ULTRA Swipes to clean & disinfect screen.

High-Touch Surfaces

Use Accel INTERVention (AHP) for:

- Bar Code Scanner/Holder
- Med Prep/Work Surface
- FESR Microphone (if used)
- Height Adjustment Handle
- Keyboard & Mouse
- Pin Pad
- Medication Drawer Handles

Deep Clean

Use Accel INTERVention (AHP) for:

- Inside of Medication Drawers
- Base



QUICK CLEAN & DISINFECT

WHAT: All high-touch surfaces
WHEN: If contaminated by direct or indirect patient contact

COMPLETE CLEAN & DISINFECT

WHAT: All exterior surfaces, including: high-touch, screen & base
WHEN: Daily/shift change; *and* on exit from an isolation room

DEEP CLEAN & DISINFECT

WHAT: By EVS, entire device including: wheels, cables, underside, inside of drawers
WHEN: Bi-weekly and daily/on discharge if dedicated to patient



Stop and Think: Is the WOW Contaminated?

- Did I use the WOW with gloved or unclean hands?
 - Did I use the WOW within two meters of splash or spray (i.e., coughing patient, suctioning)?
 - Did items/equipment from the patient room touch the WOW?
 - Was the bar code scanner put down in the patient room or in contact with the patient?
- **IF YES: QUICK CLEAN & DISINFECT**
 - **If visibly soiled with blood or body fluids, COMPLETE CLEAN & DISINFECT**
 - **If needs a deeper clean (heavily soiled, drawers, crevices, etc.), secure a replacement WOW device and arrange for DEEP CLEAN & DISINFECT by EVS**