


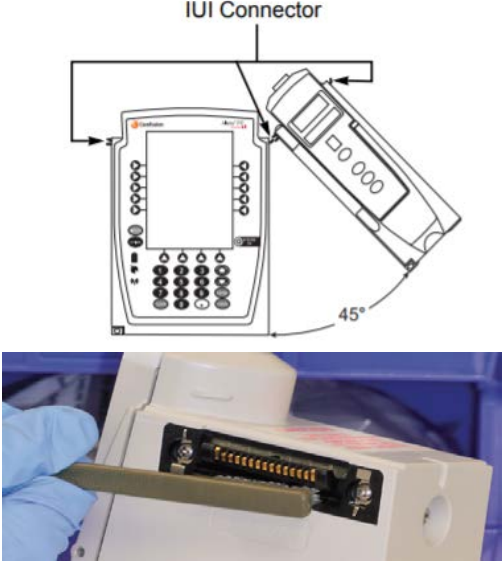


ALARIS® SMART PUMP CLEANING PRODUCTS

Key Message Use recommended products only to protect patients and pumps from damage and system failures

✓Right CLEANING PRODUCT	✓Right ALARIS® PUMP PART
<p>Disinfectant Disposable Cloths PS# 00109342</p>	<p>All exposed device surfaces Alert: except IUI connectors</p>
<p>Wipe all exposed device surfaces</p> 	
<p>Adult 30 Tuft Toothbrush: PS# 00078633 70% alcohol solution: PS# 00028305</p>	<p>IUIs Inter Unit Interface Connectors</p>
<p>Use a dedicated brush and 70% alcohol for the IUIs</p> 	
<p>Adult 30 Tuft Toothbrush: PS# 00078633</p>	<p>Case and internal parts</p>
<p>Use a different brush for cleaning the case and narrow or hard to reach areas that have visible residue.</p> <p>Alert: Don't use the same brush used to clean the IUIs</p> 