

## How to Clean and Disinfect a Workstation On Wheels (WOW) in Ambulatory and Community Settings

WHAT	WHO
<b>Quick Clean:</b> ● <b>All high-touch surfaces:</b> <ul style="list-style-type: none"> <li>barcode scanner/holder</li> <li>height adjustment handle</li> <li>FESR microphone (if used)</li> <li>medication drawer handles (if applicable)</li> <li>med prep/work surface</li> <li>keyboard &amp; mouse</li> <li>pin pad</li> </ul>	Direct Care User
<b>Complete Clean:</b> ● ◆ <b>Quick clean, plus base and screen.</b>	Direct Care User
<b>Deep Clean:</b> ● ◆ ▲ <b>Complete clean, plus all remaining surfaces.</b>	Environmental Services



WHEN	Quick	Complete	Deep
Routine – daily/shift change		◆	
Before clean task (e.g. med prep, gathering clean supplies)	●		
Used within 2 meters of a patient/client	●		
Items/equipment from patient environment touched the WOW	●		
Exiting an isolation room		◆	
Visibly soiled (e.g. small area)		◆	
Heavily soiled (e.g. large area, drawers, crevices, etc.)			▲
Routine – monthly for programs that perform invasive procedures (invasive procedures may include, but are not limited to wound care, minor surgery, endoscopy, dentistry, pelvic exams, intravaginal ultrasound)			▲
Routine – quarterly (q 3 months) for all other programs			▲

HOW (2-Step Process)	Product to Use	
<b>Step 1:</b> Clean with new wipe(s) using friction (rub/scrub).  <b>Step 2:</b> Disinfect with new wipe(s). Allow to air dry following manufacturer recommended wet contact (dwell) time for disinfection.	Accelerated Hydrogen Peroxide <b>(all surfaces except screen)</b>	Alcohol <b>(screen only)</b>
		

